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Table of contents

Preface	15
Important notes	15
Warranty	15
Declaration of conformity	15
Service life	15
Scope of supply	15
Purpose	16
Indications	16
Contraindication	16
Type plate	16
Application	16
User profile	16
Safety notes	16
General safety information	16
Information about the battery, power supply and handset	17
Installation	18
Installation of TSE 150 / TSE 150 Plus	19
Installation of TSE 150 on the eTSE	19
Installation of TSE 120 / TSE 120 Plus	20
Installation of TSE 120 on the eTSE	20
Handset holder	20
Operation and use of the eTSE	21
Charging process	22
Dismantling the eTSE	23
Cleaning	24
Maintenance and inspection	24
Reuse/processing	25
Technical specifications	25
Disposal	25
Incidents and liability	25

The eTSE toilet stand-up aid must only be used together with the Drive Medical GmbH & Co. KG products listed below, in the designated, approved combinations.

- TSE 120 with armrests
- TSE 120 Plus with armrests
- TSE 150 with armrests
- TSE 150 Plus with armrests

Preface

Thank you for choosing a Drive Medical product.

The design, functionality and quality of this product will not disappoint you.

Please read these instructions for use carefully before using the product for the first time. They contain important safety notes and valuable tips for using and caring for your product properly. If you have any questions or need additional information, please contact the Drive Medical dealer who delivered the product to you.

These instructions for use contain all the necessary information for adjusting and using the product. The latest instructions for use can also be downloaded as a pdf online or can be requested from Customer Service.

Large format documents can be displayed for visually-impaired individuals. Blind patients must be trained by their assistant.

Repairs as well as some settings require special technical training and must therefore be carried out by a specialist medical company.

Important notes

The instructions for use must be read carefully and understood before using the product.

Recommendation:

- Users, specialist staff and “laypersons” should receive product training from a specialist dealer (healthcare supply shop) before using the product.
- We recommend entering the address of your specialist dealer at the end of these instructions for use.

Warranty

The warranty period for the product we have delivered is 24 months from the date of purchase.

Provided that it can be proven, if there is a material or manufacturing fault, faulty parts will be replaced free of charge.

Parts subject to wear are generally not covered during the normal period of warranty, unless items require repair or replacement and this is necessary as a clear direct result of a manufacturing or material fault.

We reserve the right to make changes due to technical improvements and design changes.

Declaration of conformity

The technical requirements of this unit are set out in DIN EN ISO 17966. Drive Medical GmbH & Co. KG declares that this product complies with the Medical Device Regulation (EU) 2017/745.


Service life

Our company expects this product to have a service life of five years, as long as the product is used for its intended purpose and that all maintenance and service instructions are heeded.

This service life may be significantly longer, if the product is handled carefully and is serviced, looked after and used properly and there are no technical limits in the development of science and technology.

Scope of supply

However the service life can also be considerably reduced through extreme usage and incorrect use. Definition of the service life by our company does not constitute any additional warranty.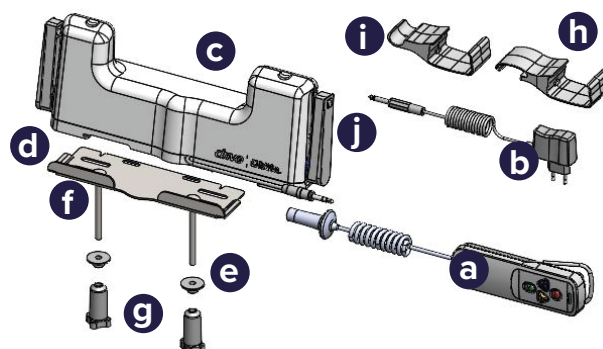

The box must contain the following components upon delivery:

- 1x eTSE handset
- 1x power supply unit
- 1x complete drive unit
- 1x base mounting plate
- 2x reducer sleeves with support collar
- 2x M8x100 carriage bolts
- 2x white knurled nuts
- 1x TSE 120 handset holder
- 1x TSE 150 handset holder
- 2x eTSE 120 rear insert adapters
- Instructions for use (not shown)

Information:

If one of the named parts is missing or damaged, please contact your dealer immediately.

Purpose

The electric toilet stand-assist device is a helpful aid that attaches to the toilet bowl and features an electric lifting and positioning function. It supports users when sitting down and standing up during toilet use. Designed for adults with limited mobility or issues affecting muscles, joints, or posture, it makes using a standard toilet easier and more independent. The lift is activated via a simple control and includes a forward motion that helps guide the user into a comfortable sitting or standing position, promoting greater independence in daily routines.

Indications


This electric toilet stand-up aid (eTSE) is designed for people who have only a limited ability to sit down and stand up when going to the toilet. In particular, this includes users with significantly reduced joint mobility and/or limited muscle strength in their lower extremities, damage to the lumbar spine, hip and/or knee joint as well as nervous system or muscle function disorders. Typical applications include patients with Parkinsons, those who have had a stroke or following hip, leg or foot surgery. As the seat can be raised, the eTSE makes it easier to sit down and stand up independently and therefore helps to maintain or regain as much independence as possible when using the toilet.

Contraindication

The eTSE may only be used on suitable, standardised toilet bowls and is only used to raise and control the height of the toilet seat. This product must not be used in any other way. The product is designed solely for adults and cannot be safely used by children, persons with a severely restricted or impaired ability to sit or those with pronounced sensory or balance disorders. The same applies if medical instructions prohibit partial or complete raising. If the weight of the user (120kg or 150kg, depending on the model of the current raised toilet seat) is exceeded, then the system must also not be used. Any warranty or liability claims are void if changes are made to the design or the product is not used correctly.

Note: The above indications and contraindications do not claim to be exhaustive. The treating doctor will decide on suitability in individual cases, weighing up potential risks and therapeutic approaches and is responsible for the recommendation to use it.

Type plate

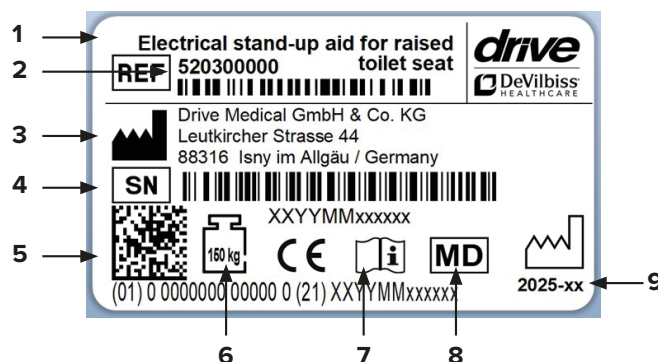1 Article description | 2 Item number | 3 Manufacturer | 4 Serial number | 5 UDI Code | 6 Max User Weight | 7 observe instructions for use | 8 Medical product | 9 production date (year/month)

Application

The electric toilet stand-up aid is used in care and nursing homes, hospitals, rehabilitation centres and private households. It must always be used with a commercially available toilet bowl and a Drive raised toilet seat (model TSE 120/ TSE 150 or TSE 120 Plus/TSE 150 Plus).

User profile

Trained nursing staff, specialist staff, patient and/or relatives who have been trained by the manufacturer/dealer (healthcare supply shop).

Safety notes

Follow these at all times! Instructions for use must be read carefully and the information contained therein must be faithfully followed! Please store these instructions for use in a safe place.

General safety information

- Do not use the device if there is a fault! Contact your dealer immediately if malfunctions occur or there is damage.
- Note that depending on circumstances, there may be a risk of slipping/falling.
- The raised toilet seat and the toilet stand-up aid must be regularly checked to ensure they are firmly in position on the toilet bowl and tighten the knurled nuts if necessary.
- Observe the maximum permissible user weight for the TSE combined raised toilet seat and electric toilet stand-up aid. This is a maximum of 150 kg for the TSE 150 and 120 kg for the TSE 120.

- Ensure that no hair, textiles or other foreign objects accumulate in the lifting mechanism. Regular cleaning prevents the formation of germs and bacteria in hard-to-reach areas.
- Always keep children and animals away from the electric stand-up aid.
- Do not make any unauthorised changes or modifications to the product. Repairs may only be carried out by approved specialist workshops or by the manufacturer. Non-compliance will render all warranty claims void.
- Do not store the product directly alongside strong sources of heat.
- The device and the handset meet the requirements of the IPX4 protection class. Whilst they are splash-proof, they are not designed for complete immersion or cleaning with high-pressure water jets. Clean the surfaces using only a slightly damp cloth. Do not spray the device or immerse it in water.

Information about the battery, power supply and handset

- Only use the handset supplied to operate the electric toilet stand-up aid.
- Never throw the handset into a fire and do not store the device near fire or strong sources of heat.
- Do not open up the handset as doing so will render all warranty claims void.
- Only use the power supply unit supplied to charge the battery.
- For your safety, the power supply unit may only be used in dry rooms and outside the bathroom. The power supply unit must only be used to charge the battery of the handset supplied.
- The battery must be charged before using the electric stand-up aid for the first time and after each use! If the battery is not used for an extended period of time, recharge it every 3 months to prolong its service life.
- Do not insert any objects that conduct electricity (e.g. knitting needles, metal pins, etc.) into the plug socket on the handset.
- If the appliance becomes unusually hot (e.g. the handset or the motor), switch it off immediately and stop using it. Contact your dealer immediately!
- If liquids or vapours leak from the handset then stop using the device immediately, avoid skin contact and inform the dealer
- Always disconnect from the mains before cleaning the handset or power supply unit.
- Use only a dry cloth to clean the handset or the power supply unit.
- Whilst both the handset and the power supply unit are sturdy, do not drop them.
- Do not use the electric toilet stand-up aid or the handset if there are visible and/or audible signs of damage. Contact your dealer in such cases. Please read the instructions in full for further information, technical details and addresses.
- These instructions contain troubleshooting information in the event of a fault. Ensure that the notes and instructions are followed faithfully.

Important safety information

- In the event of a fault, stop using the raised toilet seat and allow it to be serviced by a dealer.
- Any serious incident that occurs in connection with the device must be reported to the manufacturer as well as the competent authority of the Member State in which the user and/or patient is based.
- Only use original parts and accessories from Drive Medical GmbH & Co. KG on the product.
- Fingers, hands or objects can become trapped in the moving lifting mechanism. Keep hands and fingers away from moving parts at all times, particularly when the toilet stand-up aid is going up or down.
- Make sure that infusion hoses, catheters or other medical aids do not get caught in the lifting mechanism. Such jams may lead to injuries or blockages on the device.
- Avoid wearing long garments, belts, or scarves that could become entangled in the lifting mechanism.
- This electric toilet stand-up aid is solely provided to help patients sit down and stand up. It must not be used as a step or climbing aid! There is a considerable risk of slipping and falling when standing on the seat.
- Avoid aggressive chemicals such as concentrated bleach or strong chlorine! These can lead to flaking of the surface, corrosion or material damage.
- Use the device only after consultation with the treating doctor in order to avoid wound irritation or pain in the event of fresh wounds or recent operations in the hip/pelvic area.

Information on electromagnetic compatibility (EMC)

This product complies with the protection requirements specified in the Council Directive on the harmonisation of the laws of the Member States relating to electromagnetic compatibility (EMC, IEC 60601-1-2) (89/336/EEC).

WARNING! Risk of malfunctions due to electromagnetic faults.

In the event of electromagnetic interference, the electric toilet stand-up aid may not move up or down smoothly when the handset is operated. Electromagnetic interference may occur between this product and other electrical devices, which may impair the electrical features on this product. Under certain circumstances, these features can be disturbed or interfere with the proper operation of the electric toilet stand-up aid. If you notice any malfunctions or unusual behaviour on the electric toilet stand-up aid, or on electric devices operated nearby, stop using the product immediately and contact your dealer.

In order to prevent, reduce or eliminate such electromagnetic interference, note the following:

Only use original cables, accessories and spare parts so as not to increase electromagnetic emissions or reduce the electromagnetic immunity of this product.

Do not use this product directly next to or stacked along with other electrical appliances. Both the product and other devices must be carefully observed to ensure correct function if such use cannot be avoided.

Applied parts and classifications

Applied parts are the seat surface and armrests of the TSE 120 and TSE 150. Both applied parts are classified as type B.

Installation

During all assembly work, ensure that the armrests do not rotate/swivel unintentionally and that your fingers or hands are not trapped. Ideally, assembly should be carried out by two people; but the armrests can also be removed.

Preparations:

Remove the existing toilet seat with lid or the TSE 150/TSE 120 raised toilet seat from the toilet.

Mounting the eTSE electric toilet stand-up aid on the toilet bowl

- 1.) The illustration shows 2 reducer sleeves with support collar, 2 knurled nuts and 2 key screws (Fig. 1).

- 2.) To start, insert the two reducer sleeves with the smaller diameter from above into the holes in the porcelain toilet for attaching the toilet seat (Fig. 2).

and point upwards. The base mounting plate offers two mounting options (see illustration) depending on the desired position of the raised toilet seat (further forward or further back). Ultimately, the choice of position is also determined by the shape of the toilet. This may mean that only one assembly position can be used for the eTSE (Fig. 3).

- 4.) From above, insert the two carriage bolts through the base mounting plate and the reducer sleeves you have already inserted (Fig. 4). Make sure that the relevant

square of the carriage bolts is correctly aligned in the recess of the base mounting plate and is countersunk. Push the carriage bolt in until the mushroom-shaped head on the bolt rests on the base mounting plate.

During assembly step 5, hold the carriage bolts in this position.

- 5.) Screw the two knurled nuts onto the threads of the

carriage bolts from below and tighten them hand-tight. The conical end of the knurled nuts points upwards towards the toilet bowl. Before finally tightening the knurled nuts, check that

the mounting plate and that the carriage bolts are correctly aligned with the square in the recesses on the mounting plate (Fig. 5).

- 6.) Align the drive unit with the cable outlet pointing towards the rear of the toilet bowl. Place the drive unit centrally in the front third of the base mounting plate. Make sure that the two metal tabs on the base mounting plate are positioned in the slot-like guides on the drive housing (Fig. 6).

Now push the drive unit backwards as far as it will go. Once in the end position, the drive unit locks into place with an audible "click" (Fig. 7). After this, check that the eTSE is firmly in place.

Installation of TSE 150 on the eTSE

- 6.) Use the guide pieces described in 4.) (**Fig. 9**). Push one guide piece on the left and one on the right with the hole onto the grey pin of the drive unit as far as it will go. Use the hole on the side of the guide piece that is facing away from the clip (**Fig. 13**).
- 7.) Align both guide pieces to the shape of the housing. The clip on the guide pieces is facing upwards (**Fig. 14**).

- 8.) Place the TSE 150 on both sides with the flat guides on the assembled guide pieces and push the TSE 150 into position until it reaches a definite stop on the guide pieces. Make sure that the clip locks into position in the recess on the TSE 150 with an audible "click" (**Fig. 15 + 16**). Ensure that the TSE is in position when aligned vertically.

Installation of TSE 150 / TSE 150 Plus

Several preparatory assembly steps must be carried out before combining the actual TSE 150 raised toilet seat with the eTSE electric toilet stand-up aid. If you did not have a TSE 150 fitted on your toilet bowl before, then the dismantling steps below do not apply.

- 1.) Loosen the fixing screws on the TSE 150 raised toilet seat, then remove the seat from the toilet bowl.


- 2.) Press the clip slightly inwards on one side until it unlocks. Pull the guide piece about 1cm out of the guide so that the clip does not lock back into place (**Fig. 8**).

- 3.) Repeat this procedure on the opposite side and now pull both guide pieces, including the transverse tube, completely out of the guides. Finally, pull both guide pieces out of the tube.

- 4.) The two dismantled guide pieces (**Fig. 9**) will be required later to link the TSE 150 with the eTSE toilet stand-up aid (see point 6). Mounting the TSE 150 on the eTSE.

- 5.) Positioning the transverse tube at the front.
 - Pull both **front** flat guide pieces, together with the transverse tube, fully out of the flat guide (there is no locking clip) (**Fig. 10**).
 - Pull both flat guide pieces out of the transverse tube. Press the transverse tube on both sides into the relevant hole on the flat guide pieces for the **highest** setting level down to the stop (**Fig. 11**).
 - Place the preassembled transverse tube with both flat guide pieces back into the flat guide and press this down as far as the base on the flat guide (**Fig. 12**).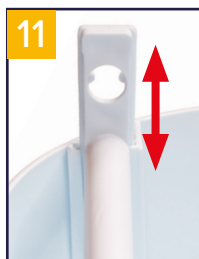

Installation of TSE 120 / TSE 120 Plus


Several preparatory assembly steps must be carried out before combining the actual TSE 120 raised toilet seat with the eTSE electric toilet stand-up aid. If you did not have a TSE 120 fitted on your toilet bowl before, then the dismantling steps below do not apply.

- 1.) Loosen the fixing screws on the TSE 120 raised toilet seat, then remove the seat from the toilet bowl.

- 2.) Press the clip slightly inwards on one side until it unlocks. Pull the guide piece about 1 cm out of the guide so that the clip does not lock back into place (**Fig. 17**).

- 3.) Repeat this procedure on the opposite side and now pull both guide pieces, including the plastic transverse holder, completely out of the guide (**Fig. 18 + 19**).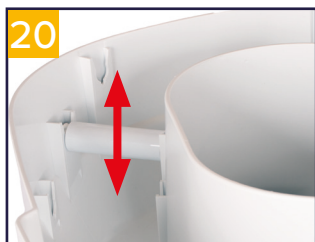

- 4.) Put the two front supports in the top position in this step. To do so, place the TSE 120 on the seating area and pull both cylindrical supports out of the respective holders. Place both front supports, as shown in the illustration, in the top position and press them down as far as they will go in the guide (**Fig. 20**).

shown in the illustration, in the top position and press them down as far as they will go in the guide (**Fig. 20**).

Installation of TSE 120 on the eTSE

- 5.) Use both **supplied** guide pieces that are delivered with the eTSE for further assembly (page 3, item j). Push one guide piece on the left and one on the right with the hole onto the grey pin as far as it will go. Align both guide pieces to the shape of the housing. The clip on the guide pieces is facing upwards (**Fig. 21**).

- 6.) Place the TSE 120 on both sides with the flat guides on the assembled guide pieces and push the TSE 120 into position until it reaches a definite stop on the guide pieces. Make sure that the clip locks into position in the recess on the TSE 120 (**Fig. 22 + 23**).

Handset holder

The eTSE includes separate holders for the handset on the TSE – version 150 (**Fig. 24**) or 120 (**Fig. 25**). Assembly of the components is described below.

The holder for the handset can be attached to the left or right-hand armrest as required.

Mounting the holder for the handset on the TSE 120 (Fig. 26)

Place the relevant holder (Fig. 25) on the rear end of the armrest on the left or right-hand side and align it with the guide geometry on the armrest.

Push the holder in the direction of the front end of the armrest and position as you require.

Put the handset into the holder provided when not in use.

Mounting the holder for the handset on the TSE 150



Select a side for mounting the handset.

Unscrew the screw at the front of the armrest fully in an anti-clockwise direction (Fig. 27).

Attach the holder supplied (Fig. 24) for the handset using the screw on the armrest that was taken out beforehand. Screw the screw in fully in a clockwise direction until it is hand-tight.

Make sure that the holder is aligned with the outward-facing side of the armrest (Fig. 28).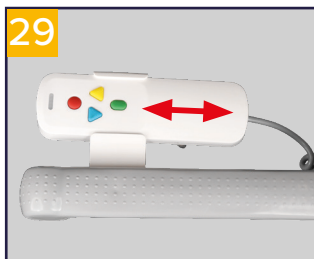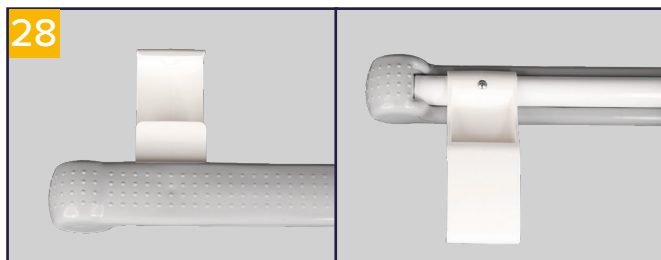

Put the handset into the holder provided when not in use (Fig. 29).

The handset is required to control the upwards/downwards movement of the eTSE. Connect these to the eTSE by inserting the jack plug on the eTSE into the plug socket of the handset as far as it will go.

Operation and use of the eTSE

CAUTION – Risk of falling and crushing due to improper seating position

Always sit centrally on the seat of the mounted toilet seat riser.

Avoid uneven weight distribution (e.g., sitting predominantly on one side) and sitting too far back.

Place both feet flat on the floor and use both armrests symmetrically when sitting down and standing up.

Do not put weight on the edges (front/back) and do not lean exclusively on one armrest.

Description of functions for the buttons on the handset

Green button: Pressing this button switches the handset on and it is ready to use.

Red button: Pressing this button switches the handset off.

STOP function: The red button has an additional function. If a fault occurs during operation, pressing the red button stops all movement immediately and switches the handset off.

Yellow button: Pressing this button raises the adapted raised toilet seat.

Blue button: Pressing this button lowers the adapted raised toilet seat.

Various operating modes and the battery charge state are shown via the LED display on the handset. The LEDs illuminate or flash in yellow, red, green and grey (not illuminated).

Possible LED colour combinations and their meaning when in motion

Function	Display LED colours			
Moving mode	LED 1	LED 2	LED 3	LED 4
Battery capacity is 100%, handset ON LED 1 – 4 light up				
Battery capacity is 75%, handset ON LED 1 – 3 light up, LED 4 off				
Battery capacity is 50%, handset ON LED 1 and 2 light up, LED 3 and 4 off				
Battery capacity is 25%, handset ON LED 1 lights up, LED 2 - 4 off				
Battery capacity is low, only one further moving cycle possible, LED 1 – 4 flash yellow				
Battery capacity is low, only one further UPWARDS lift function possible, LED 1 – 4 flash red				
Error display for short circuit or power consumption error. LED 1 & 4 off. LED 2 & 3 light up				
System error for hardware or firmware LED 1 and 4 light up, LED 2 and 3 off				
OFF mode, LED 1 - 4 are off				

Charging process

Only the plug-in power supply unit included as part of delivery may be used to charge the rechargeable battery in the handset. The voltage specification on the power supply unit must match that of the public mains supply.

Various operating modes and the battery charge state are shown via the LED display on the handset. The LEDs illuminate or flash in yellow, red, green and grey (not illuminated).

To charge the battery, proceed as follows:

- 1.) Insert the jack plug on the plug-in power supply unit into the jack plug socket provided on the spiral cable end of the handset until it engages in the socket.
- 2.) Plug the power supply unit into a socket for charging.
- 3.) As soon as charging is complete, first unplug the power supply unit from the socket, then remove the jack plug from the jack plug socket.

Note: If the charging indicator does not light up after connecting the power supply unit to the handset, please check the following:

- Is it plugged in correctly
- Check all components involved for damage
- Check that the wiring is supplied with power (check house wiring)

Contact your dealer if one of the components is damaged.

Possible LED colour combinations and their meaning when in charging mode

Function	Display LED colours			
Charging mode	LED 1	LED 2	LED 3	LED 4
Battery charged to 100% capacity (LED 1 – 4 light up)				
Battery charged to 75% capacity, is being charged to 100% capacity (LED 1 – 3 light up, LED 4 flashes)				
Battery charged to 50% capacity, is being charged to 75% capacity (LED 1 & 2 light up, LED 3 flashes, LED 4 is off)				
Battery charged to 25% capacity, is being charged to 50% capacity (LED 1 lights up, LED 2 flashes, LED 3 & 4 are off)				
Battery is being charged to 25% capacity (LED 1 flashes, LED 2 – 4 are off)				
An error has occurred during the charging process (LED 1 & 4 light up red, LED 2 & 3 are off)				
The battery pack is not fully charged after 6 hours (LED 1 & 4 are off, LED 2 & 3 light up red), error mode				

Dismantling the eTSE

If it is necessary to remove the TSE 120/150 and the eTSE from the toilet then please follow the steps stated below. The process is the same for both raised toilet seats.

- Press the plastic clips inwards on both sides and remove the body of the seat to take the adapted raised toilet seat off.
- Pull the seat body up and away from the guide pieces and then disconnect the toilet seat raiser from the electric stand-up aid.
- There are two recesses (width approx. 15 mm) in the plastic housing at the front of the eTSE. Push the blade of a screwdriver, width 6 - 8 mm and thickness of approx. 1 mm, into this recess (**Fig. 30+31**).

- There is a stop at a depth of approx. 7mm. Lower the screwdriver handle carefully downwards with a little pressure in the direction of the eTSE. As soon as it is in the correct position, the screwdriver can be pushed a further 3 mm or so in the direction of the eTSE.
- When at the correct insertion depth, lower the screwdriver handle by around 10 to 20 degrees in the direction of the floor. A clip is now unlocked in this position. From the rear, push the eTSE forwards by 2 - 3 mm on this side so that the clip cannot lock into position again.
- Pull the screwdriver out and repeat the same process on the opposite side. Hold the unlocked side firmly at the front so that this does not spring back into position when the other side is unlocked, therefore unintentionally locking it again.

Now move the eTSE left and right simultaneously towards the front of the toilet. Remove the unit from the toilet and store in a safe place.

Address of the dealer:

Cleaning

- Use a slightly moistened cloth to clean the eTSE and its handset.
- Make sure the power supply unit is disconnected from the socket before cleaning it with a cloth.
- If the unit is heavily soiled, use commercially available cleaners and a brush if necessary.
- Do not clean the units with strong cleaners (containing chlorine or acetic acid), do not use scouring agents.
- Do not immerse any of the components in water or any chemical cleaning fluids.

Caution: If the electric toilet stand-up aid needs to be cleaned and disinfected in a long-term care facility or a clinical environment:

Follow the manufacturer's preparation and hygiene guidelines (Drive Medical GmbH & Co.KG) as well as in-house procedures.

Disinfection:

WARNING! The electric toilet stand-up aid must be disinfected before each change of user. There is a risk of infection. The manufacturer's usage and handling instructions for cleaning and disinfection agents should be observed during disinfection.

Recommendations for disinfectants:

- Descosept AF
- Bacillol AF

Maintenance and inspection

CAUTION: Servicing and inspection must not take place during use.

Check that the device is securely attached to the toilet bowl before each use; if necessary, retighten the nuts.

Carry out a visual inspection for external damage before each use.

If there is a malfunction or visible signs of wear then contact an approved dealer immediately.

An approved dealer must carry out any necessary repairs.

Recommendation: An approved dealer should carry out a visual and functional inspection of the electric stand-up aid at least every 2 years.

The power supply unit is maintenance-free.

Reuse/processing

- The electric toilet stand-up aid must be disinfected before each change of user. This is important as it prevents cross-contamination. There is a risk of infection.
- This product is designed to be re-used.
- Make sure that all technical documentation required to ensure safe usage is passed on to the new user.
- The product must be checked and cleaned by the retail trade who will restore it to perfect working order ready for re-use.
- When preparing the product, the manufacturer's instructions must be observed and these must be made available on request.

Ambient conditions for operation

Temperature	+10 °C – +35 °C
Relative humidity	15 – 90 %
Air pressure	700 hPa – 1060 hPa

Ambient conditions for transport and storage

Temperature	-10° C – +40 °C
Relative humidity	10 – 100 %
Air pressure	700 hPa – 1060 hPa

Technical specifications

Length	87 mm
Width	406 mm
Height	158 mm
Vertical stroke	95 mm
Weight of handset	450 g
Weight eTSE	3,800 g
Max. Load	150 kg
Length of motor cable	405 mm
Nominal voltage of battery	11.1 V

Material

Drive unit cover	PA + 30 % glass fiber
Manual control unit	ABS

Disposal

If you no longer require the product then please contact your dealer or take the product to your local disposal site.

This medical device contains a rechargeable lithium-ion battery. The device and the battery must be collected separately from household waste. Lithium-ion batteries must not be thrown into fire. Improper disposal can lead to fires, explosions, and environmental damage. Lithium-ion batteries must be taken to an authorized collection point. You are legally required to return batteries in accordance with national regulations. The battery must be completely discharged, and its terminals must be taped prior to disposal.

Disposal and recycling of old units and packaging must take place in accordance with current legal requirements. Devices covered by the German Electrical and Electronic Equipment Act are listed as medical products by the "Elektro-Altgeräte-Register" for professional B2B use. Our registration number is DE 33275322.

Incidents and liability

Drive Medical GmbH & Co. KG only accepts liability if the products are used under the conditions and for the purposes specified by us. Only approved dealers or the manufacturer may carry out repairs on the product.

In the event of incidents involving the product or personal injury with one of our products, please contact your dealer immediately. They will then contact us as the manufacturer and we will handle the case.

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